MEISINGER

Handpiece *Experts* Certified

BUR CATALOGUE

READING & FORMULATING BUR NUMBERS

Fig.	Shank	ISO-No.		5	5	5	5	5
801L	HP	806 104 697 524		008	010	012	014	016
	FG	806 314 697 524			010	012	014	016
801LG	FG	806 314 697 534			010	012	014	016

801L	G	-	012	FG
1	2		3	4

1 Bur Model Number (80I, 80IL, 389, etc.)

2 Bur Grit

Ultra Coarse	Super Coarse	Coarse	Medium	Fine	Extra Fine	Ultra Fine
S	H	G	(leave blank)	F	C	U

3 Head Diameter (mm) (012, 014, 016, etc.)

4 Shank Type (See p.XX of the Meisinger Catalogue)

ORDERING BURS FROM HANDPIECE EXPERTS

1 Fill bur order sheet (download bur order sheet from www.wikihpe.com)

Before faxing the sheet, make sure:

- That you included your dealer number and/or your contact information
- That you are ordering burs in the correct quantities (see the next page)
- That you included your customer address if you selected drop-shipping

2 Fax your bur order sheet to 1-866-285-1171 or email it to orders@handpieceexperts.com

Note: To avoid mistakes due to poor handwriting, you can download and edit the electronic copy directly and then print and fax, or email it to our orders team.

3 Once the order is treated, you will receive a fax or email confirmation.

Note: Bur orders can be complex: We recommend you review your order before you place it and after receiving confirmation.

4 The burs you ordered will then be delivered directly to you or your customer.

> Shipping

A flat ground shipping charge of \$10 will be added to every order.
Free shipping for orders over \$250 does not apply to Meisinger bur orders.
Priority shipping is not available at this time.

> Payment Method

For credit card payments, we accept: American Express, Visa, and MasterCard.
On 30 day payments you will need to fill out a credit application.
Orders under \$500 do not require a credit application – this only applies if your balance is 0.

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Many ranges of the rotary instruments, e.g. fitting dimensions with shaft diameter and shaft size as well as size information have already been internationally standardized and uniformly marked by the ISO number system. Together with the fixed number code of the ISO-order number a great amount of information such as form, size and character can be obtained which distinctly enables the unique identity of the instrument.

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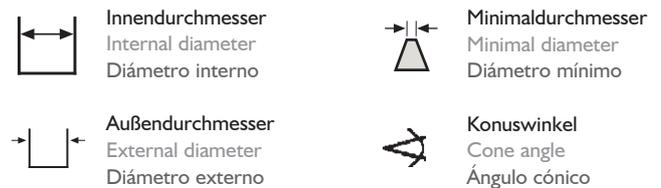
In rare cases the packing unit can vary within the chart index. Please take the exact numbers of pieces per packing unit from the price list.

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Application and hygiene symbols for dental instruments

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Implantologie
Oral implantology
Implantología

Stiftsystem
Implant pin system (post system)
Sistema de anclaje con espiga

Bone Management
Bone Management
Bone Management

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The symbols give merely suggestions for the possible implementation of the products. The user decides and takes full responsibility about the precise deployment according to existing indications. Please take notice of our current Application and Safety Instructions. Details can be found on page 230, as well as on the internet under www.meisinger.de, or you can request for one by post.

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Types of shanks and overall lengths

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Digit 4 to 6 of the ISO numbering system identify the type of shank (diameter, overall length, denomination):

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* The total lengths of instruments can be shorter or longer according to type of construction.

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Best results with rotary instruments by MEISINGER are obtained at following suggested speeds min^{-1} (R.P.M.). The suggested speeds are based non technical requirements (construction and material).

Symbole – Beispiel:
 Symbols – example:
 Symboles – exemple:
 Símbolos – ejemplo:

= nicht über 120.000 min^{-1} (UpM)
 = not exceeding about 120.000 min^{-1} (R.P.M.)
 = ne pas dépasser environ 120.000 min^{-1} (T.p.m.)
 = no superior a aprox. 120.000 min^{-1} (R.p.m.)

Grösse Size Taille Tamaño	Praxis Dentistry Cabinet Clinica		Chirurgie Surgery Chirurgie Cirurgia					
	Hartmetall-/Diamant Tungsten Carbide-/Diamond Carbure-/diamantés Carburo (Tungsteno)-/diamante		Hartmetall-/Diamant Tungsten Carbide-/Diamond Carbure-/diamantés Carburo (Tungsteno)-/diamante					
	FG	Winkelstück	Handstück	Winkelstück				
	FG FG FG	RA CA CA	HP PM PM	RA CA CA				
	min^{-1}	m/sec	min^{-1}	m/sec				
005	300.000	8	160.000	4				
006	300.000	9	160.000	5				
007	300.000	11	160.000	6				
008	300.000	13	160.000	7				
009	300.000	14	160.000	8				
010	300.000	16	160.000	8	50.000	2,6	50.000	2,6
012	300.000	19	160.000	10	45.000	2,8	45.000	2,8
014	300.000	22	160.000	12	35.000	2,6	35.000	2,6
016	290.000	23	160.000	13	32.000	2,7	32.000	2,7
018	250.000	24	160.000	15	28.000	2,6	28.000	2,6
021	210.000	23	160.000	18	25.000	2,7	25.000	2,7
023	190.000	23	160.000	19	22.000	2,6	22.000	2,6
025	190.000	24	120.000	16	20.000	2,6	20.000	2,6
027	160.000	23	120.000	17	18.000	2,5	18.000	2,5
029	150.000	23	120.000	18	16.000	2,4	16.000	2,4
031	150.000	24	120.000	19	16.000	2,6	16.000	2,6
033	120.000	21	120.000	21	15.000	2,6	15.000	2,6
035	120.000	22	120.000	22	15.000	2,7	15.000	2,7
037	120.000	23	120.000	23	14.000	2,7	14.000	2,7
040	100.000	21	100.000	21	13.000	2,7	13.000	2,7
042	100.000	22	100.000	22	13.000	2,9	13.000	2,9
045	80.000	19	80.000	19	11.000	2,6	11.000	2,6
047	80.000	20	80.000	20	11.000	2,7	11.000	2,7
050	80.000	21	80.000	21	10.000	2,6	10.000	2,6
055	80.000	22	80.000	22	10.000	2,8	10.000	2,8
060	60.000	19	60.000	19	9.000	2,8	9.000	2,8
065	60.000	20	60.000	20	8.000	2,7	8.000	2,7
070	60.000	22	60.000	22	7.000	2,6	7.000	2,6
075	50.000	20	50.000	20	6.500	2,6	6.500	2,6
080	50.000	21	50.000	21	6.000	2,5	6.000	2,5
085	45.000	20	45.000	20	6.000	2,7	6.000	2,7
090	45.000	21	45.000	21	5.500	2,6	5.500	2,6
095	45.000	22	45.000	22	5.500	2,7	5.500	2,7
100					5.000	2,6	5.000	2,6
110					4.500	2,6	4.500	2,6
120					4.000	2,5	4.000	2,5
130					4.000	2,7	4.000	2,7
140					3.500	2,6	3.500	2,6
150					3.500	2,7	3.500	2,7
160					3.000	2,5	3.000	2,5
170					3.000	2,7	3.000	2,7
180					2.500	2,4	2.500	2,4
190					2.500	2,5	2.500	2,5
200					2.500	2,6	2.500	2,6
220					2.300	2,6	2.300	2,6

Empfohlene Richtdrehzahlen (Schnittgeschwindigkeitsbereiche)
Suggested Speeds (Speed Ranges)
Vitesses recommandées (portées des vitesses moyennes)
Velocidades recomendadas (alcance de velocidades de corte)

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Diamond Instruments

MEISINGER diamond instruments consist of a shank and a hardened, stainless profile body coated by means of the most modern galvanisation process, with selected diamond grains. This guarantees an extremely homogenous, safe and lasting diamond coating of the instruments and thus optimal working results. In addition, the user is guaranteed a choice from up to seven different grain sizes.

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The use of the coarse grain diamond (ISO 534, 544 and 554) can lead to increased thermal development. Therefore, when using these products one should take particular note of adequate cooling and minimal applied pressure. After attaining ideal rough depths with the diamond instrument, the subsequent use of a finishing bur is essential.

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Tip not diamond coated, for gentle administration of the cervical area

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
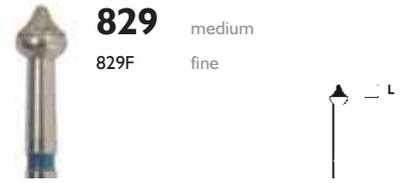
831

medium
831H super coarse
831G coarse
831F fine
831C extra fine
831U ultra fine

Fig.	Shank	ISO-No.		5	5	
				L mm	3,5	3,5
831	FG	806 314 254 524		016	018	
831H	FG	806 314 254 544		016	018	
831G	RA	806 204 254 534		016		
	FG	806 314 254 534		016	018	
831F	FG	806 314 254 514		016	018	
831C	FG	806 314 254 504		016	018	
831U	FG	806 314 254 494		016	018	

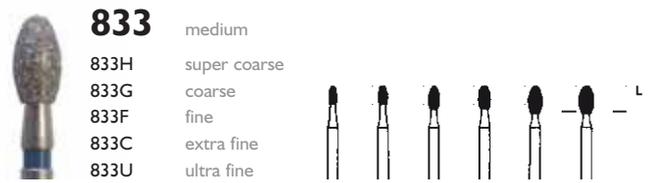
833

medium
833H super coarse
833G coarse
833F fine
833C extra fine
833U ultra fine

Fig.	Shank	ISO-No.		5	5	5	5	5	5	
				L mm	2,8	2,8	3,4	3,4	4,2	4,2
833	FG	806 314 277 524			014	016	018		023	
833H	FG	806 314 277 544			014	016	018		023	
833G	FG	806 314 277 534			014	016	018		023	
833F	RA	806 204 277 514							023	
	FG	806 314 277 514		012	014	016	018	021	023	
833C	FG	806 314 277 504			014	016	018		023	
833U	FG	806 314 277 494			014	016			023	

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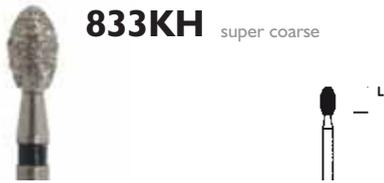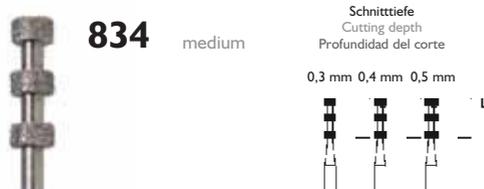
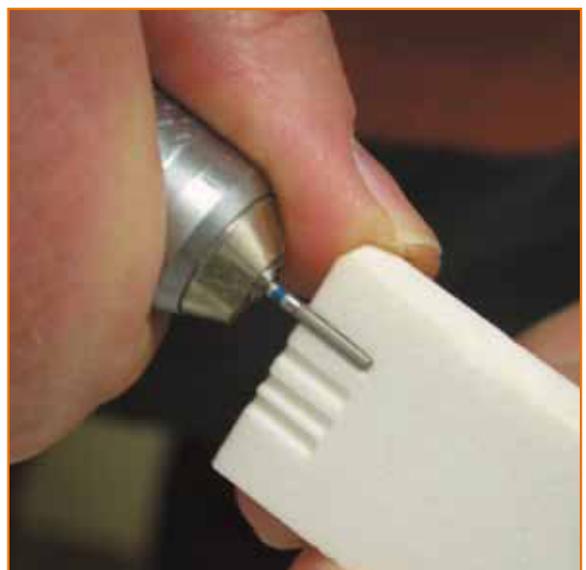
Designed for fast and easy cleaning of diamond instruments, for improved performance (cleaning stone to be moistened thoroughly before use).

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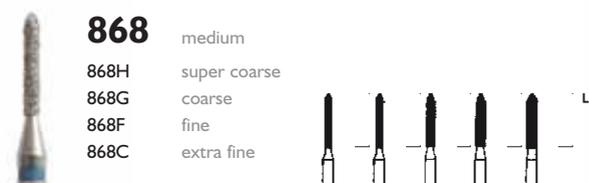
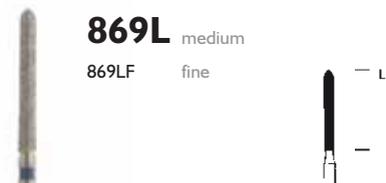
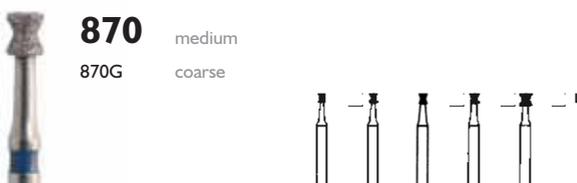
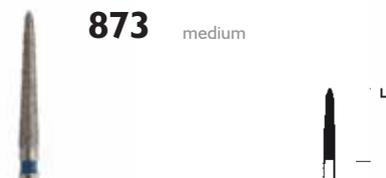
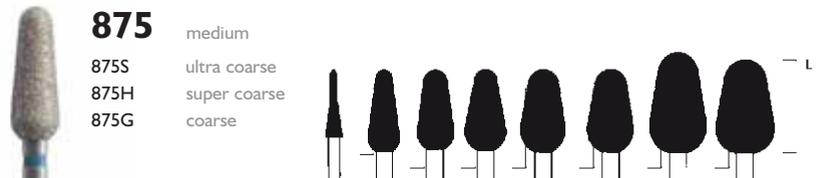
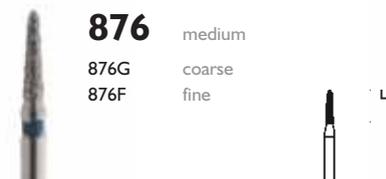
Advantages 881PS

- For grinding soft underlining material
- Diamond coated with stronger, sharp edged grit
- New structure prevents clogging effect of the sharp edged grit
- Reduced heat development

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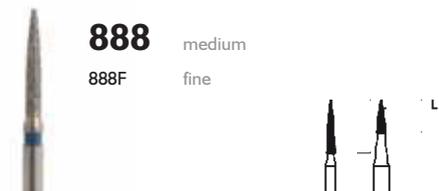
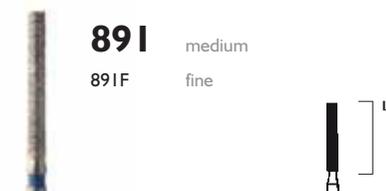
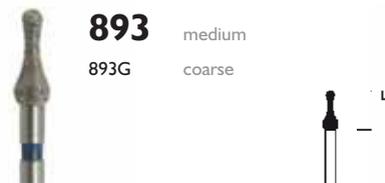
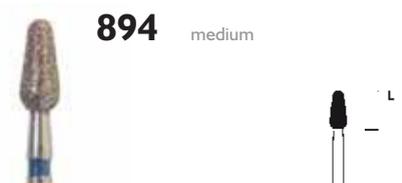
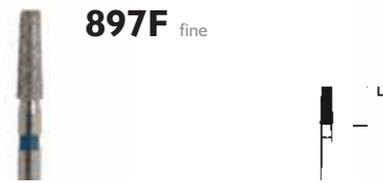
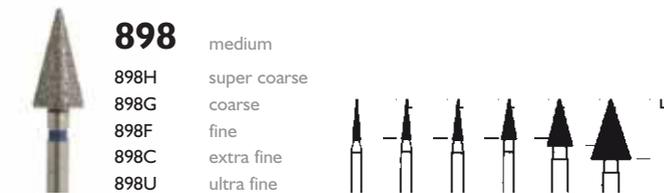
Dual controlled groove grinder - according to Küpper

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- System to achieve a simple and controlled shoulder preparation
- Supported, secure depth marking (cutting depth 4.0 mm, 6.0 mm, 8.0 mm)
- Minimal loss of substance due to torpedo-shaped diamond
- Excellent use also during the filling stage due to the slight raising of the occlusal surfaces
- Simple controlled procedure
- Makes work much easier and delivers perfect end results

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Visit our Website:
www.occlusalfrouter.com
 U.S. Patent #6,511,322
 Foreign Patent Pending

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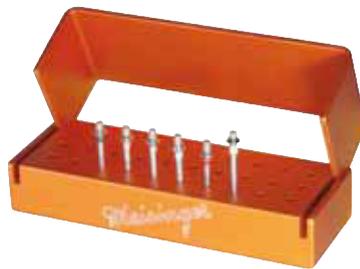
The Occlusal Reduction grinder has a patented diamond design which allows controlled access to cusps and fissures of the tooth easily from a vertical direction. Thanks to the unique angled dead stop the diamond is limited to its working depth.

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2565 Occlusal Reduction Kit

OCCLUSAL REDUKTION

Depth	Indications
1.5 mm	Gold crowns
1.8 mm	Short clinical crowns - ceramic or all ceramic
2.0 mm	PFM crowns, CEREC®, Captek, or all ceramic
2.2 mm	Long clinical crowns - PFM or all ceramic
2.4 mm	To level the occlusal plane

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The newly developed MEISINGER diamond with an integrated, guided cooling-system

The non-diamond, spiral cooling canals of the MEIPREP diamond instruments guarantee non-traumatic, fast and efficient work. Through intermittent grinding and an optimal permanent cooling of the preparation site, possible heat build-up is avoided right from the start. The super coarse (180 micron) MEIPREP diamond instruments guarantee highly efficient cutting and reduced preparation time. The large variety of designs makes almost all dental preparations possible.

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CE diamond instruments (COOL & EFFICIENT) provide for optimum grinding performance yet remain cool during grinding. These positive properties are due to a precisely calculated, anti-clockwise groove which transports the debris away from the site and supplies coolant optimally.

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Advantages:

- Maximum preservation of sound tooth substance
- Perfect optical control of the working area
- Defined abrasive properties
- Vast spectrum of operational possibilities due to a variety of forms

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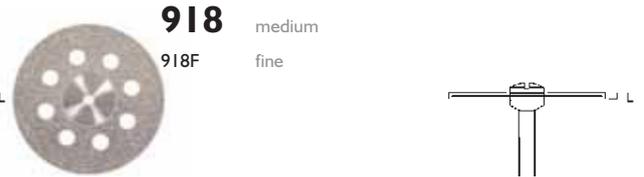
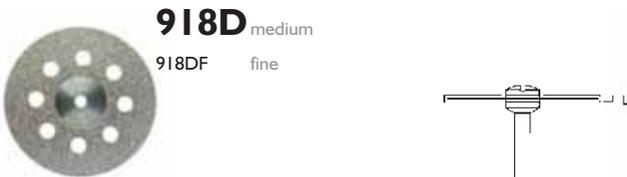
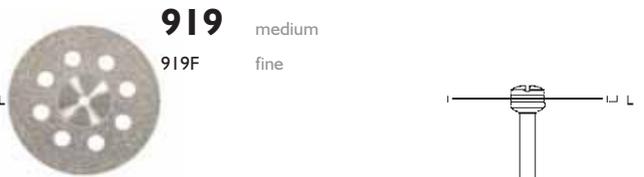
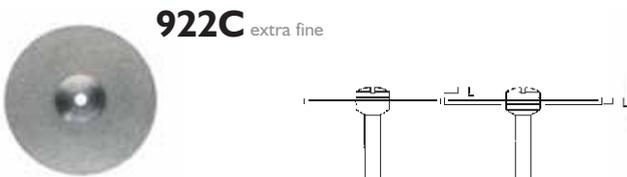
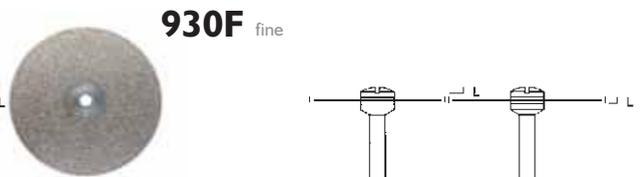
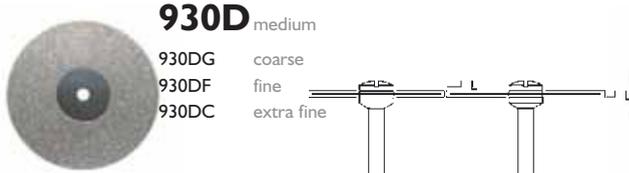
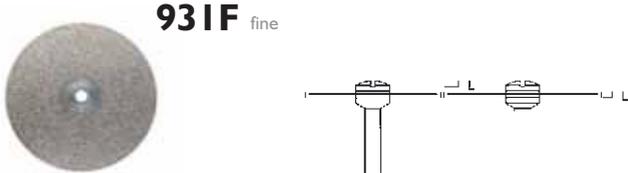
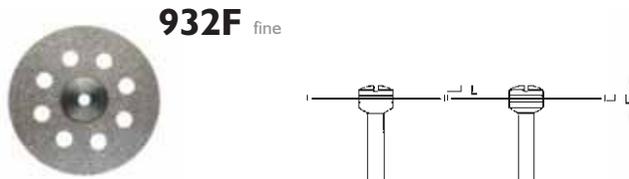
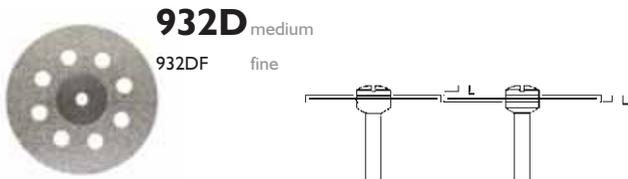
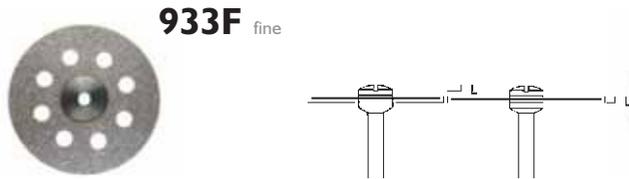
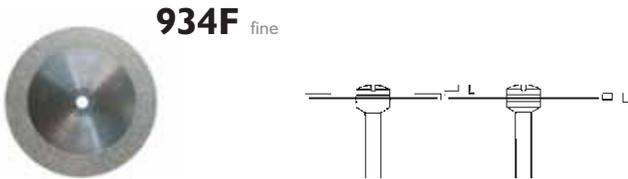
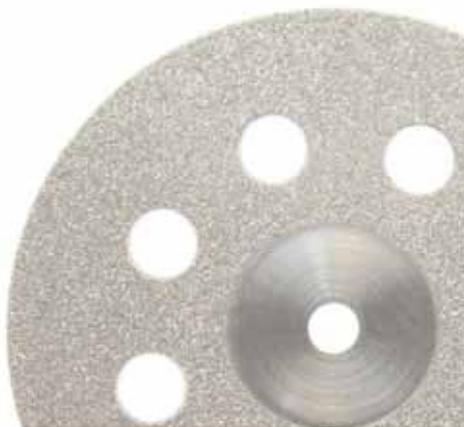
Recommended usage for diamond steel brushes:
Speed range: 5,000–10,000 rpm pressure approx. 0.5 N (50 g).
Light pressure will give optimum results. Avoid sudden changes
in speed. Use brushes only in one rotating direction (fig. 991C
for clockwise direction only).

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MEISINGER sintered diamonds are characterized by their extremely long endurance and dimensional stability as well as excellent abrasive qualities. The working part consists of a concentrated and homogeneous combination of exclusively selected, top quality diamond grains sintered and bound in bronze. Sintered diamond instruments are available in three grit sizes (coarse, medium and extra fine).

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Duroflex - highly flexible; specifically for the separation of porcelain and ceramics, without causing discolouration

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The sintered diamond disc in size 080 is especially suitable as crown cutter for non precious metals

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Duroflex - split - highly flexible; specifically for the separation of porcelain and ceramics, without causing discolouration

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To obtain best results, the sintered diamond cleaning stone should be used dry

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Duroflex-split: highly flexible; specifically for the separation of porcelain and ceramics, without causing discolouration

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Tungsten Carbide Instruments

MEISINGER tungsten carbide instruments are compelling due to their functional precision toothing, above average concentricity, outstanding efficiency and long endurance. They are made of high density fine grain metal and are characterized by their practical dimensions.

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- Greatly reduced vibration
- Optimal revolving properties
- Fast excavation as a result of new cutting geometry
- Minimal feeding pressure
- Easy recognition by means of blue color ring

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Precision toothing and SQ cut for excavation, with an extra slender neck design for an ideal view on the application area as well as an enhanced coolant supply.

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Sharp edged undercut can produce a damaging notch effect!

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Amalgam removal HM 31A, Crown cutter HM 31C and HM 33C, developed with University RWTH Aachen

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The SPECIALIST in removal of amalgam
 The newly developed amalgam removal tool with its practical working section length reduces the emergence of toxic mercury fumes. The special radial cut and the fine cross-cut facilitate axial preparation. This leads to a fine grinding of splinters and very high cutting performance.

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The SPECIALIST in separating metal and ceramic
 The special radial cut of the instrument's working section and the optimal drilling properties guarantee easy separation of crowns and bridges of all alloys and ceramic veneer. The coarse principal cut in combination with a helical cross-cut result in high cutting performance and fracture resistance.

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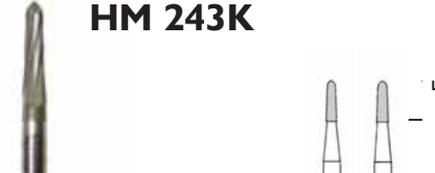
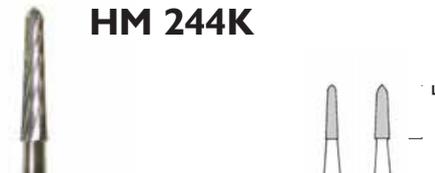
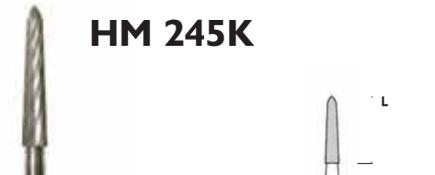
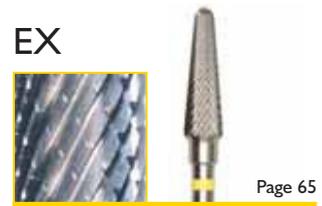
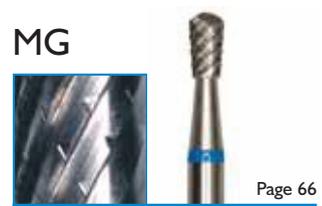
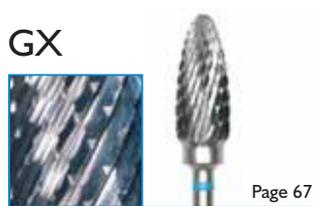
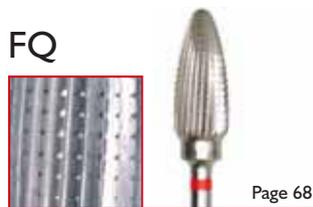
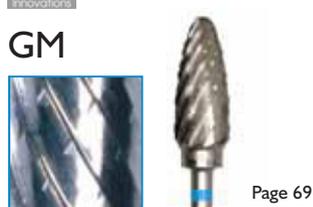
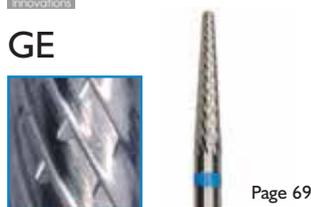
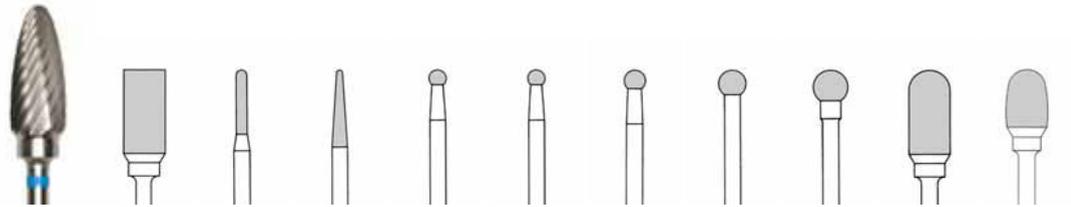
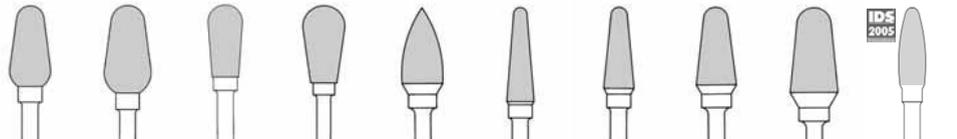
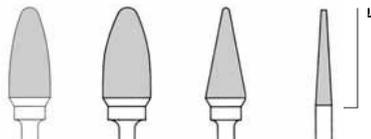
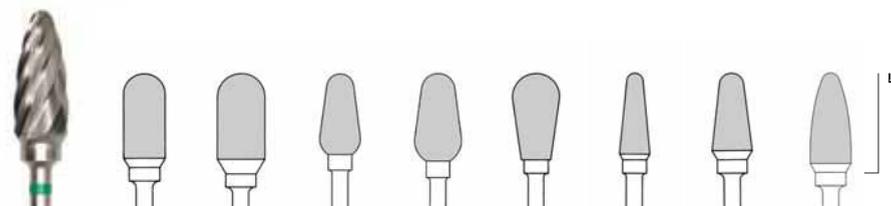
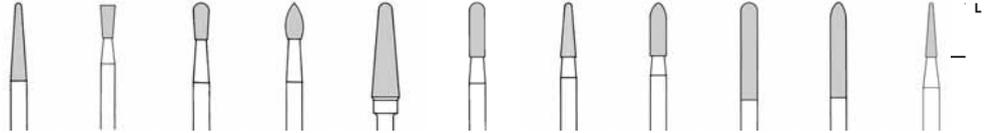
X-cut, extra fine (ISO-No. 141), yellow ring, for fine, precise trimming of ceramics, precious metals and all non-precious alloys

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
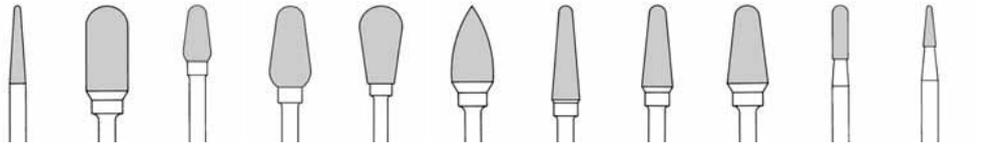
X-cut, fine (ISO-No. 140), red ring, for delicate and difficult trimming of hard alloys, acrylics and ceramics

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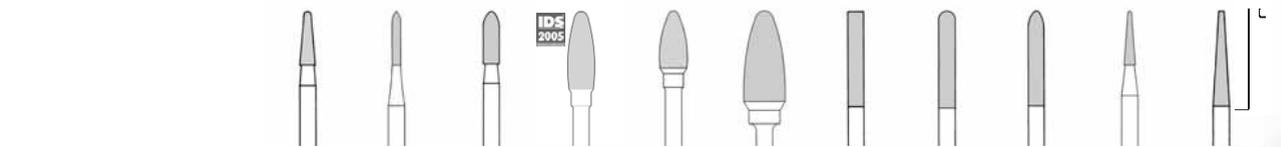
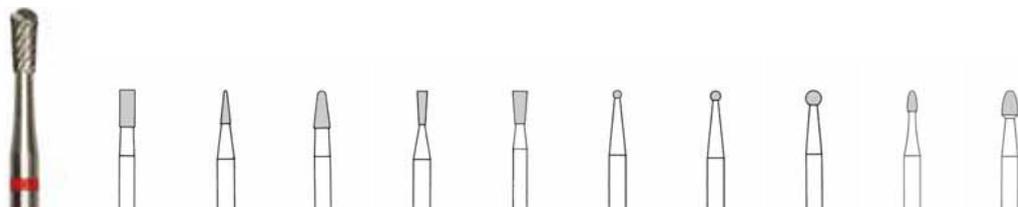
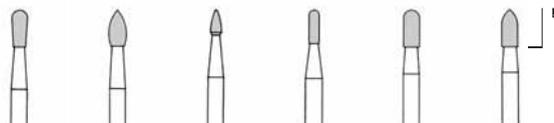
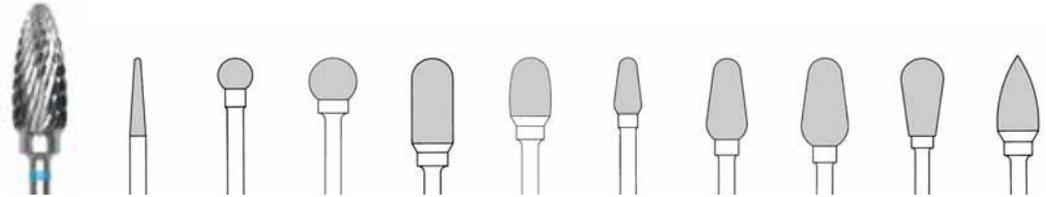
X-cut, medium (ISO-No. 190), blue ring, for trimming large surfaces of non-precious alloys and acrylics

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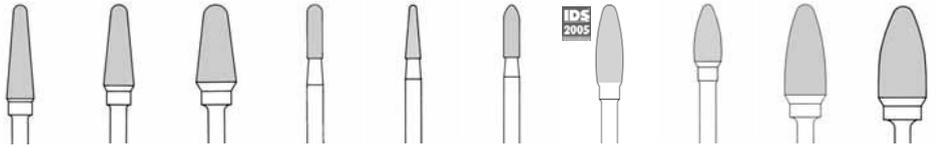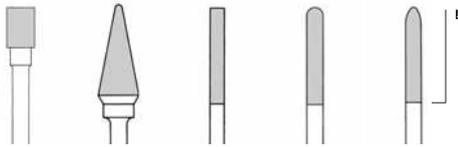
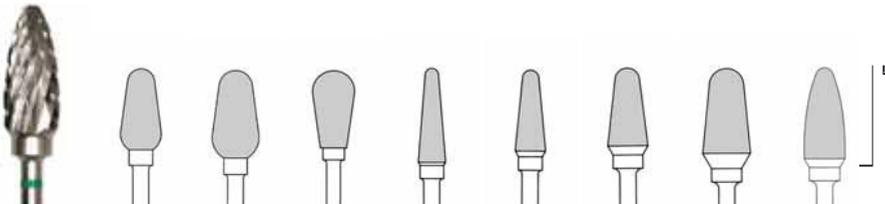
X-cut, coarse (ISO-No. 220), green ring, for trimming large surfaces of acrylics, tray materials and plaster

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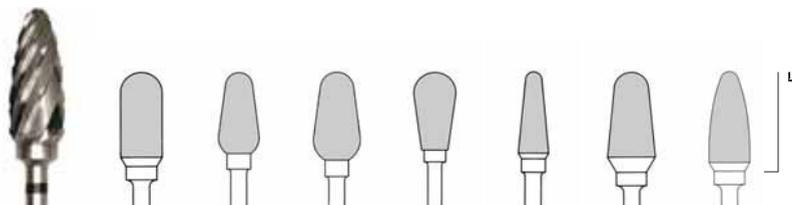
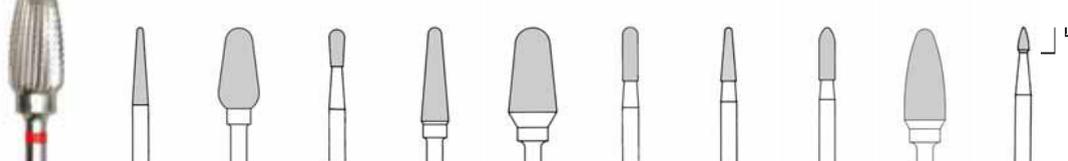
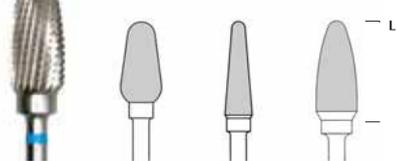
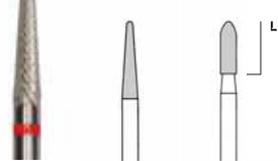
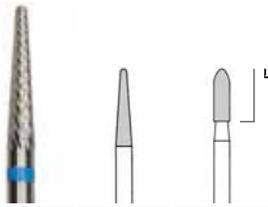
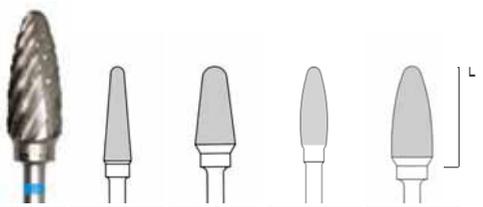
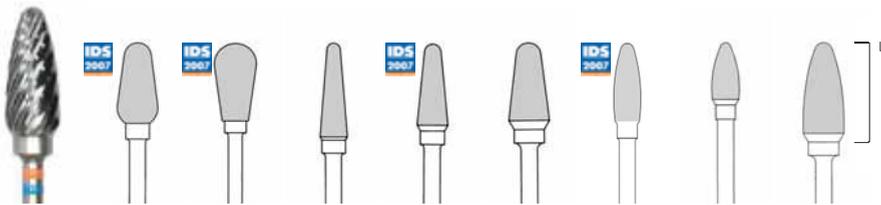
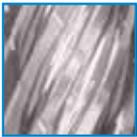
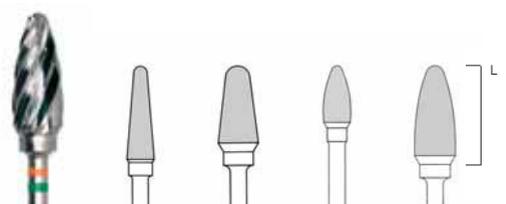
- Reinforced, resistant and widened beveled cutters
- Ample chip spaces, for the expulsion of more material
- Especially suitable for all permanent soft acrylic resins and other acrylics, as well as for gypsums
- In spite of high material removal a good surface quality is reached
- Also in high rotation speeds, very high out-put (up to 30.000 min⁻¹)

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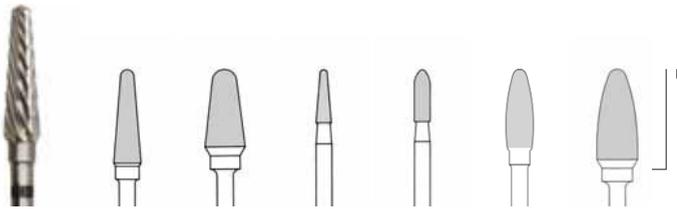
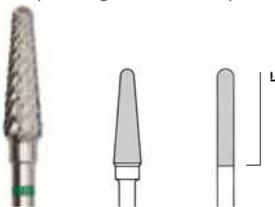
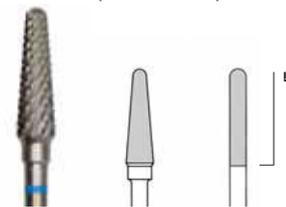
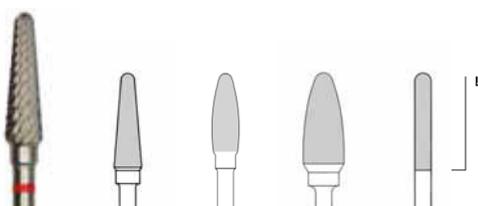
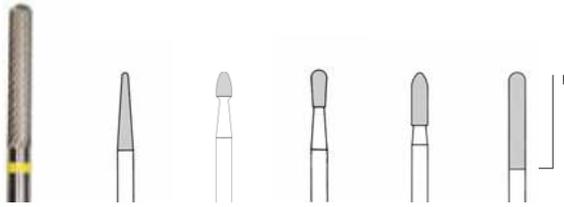
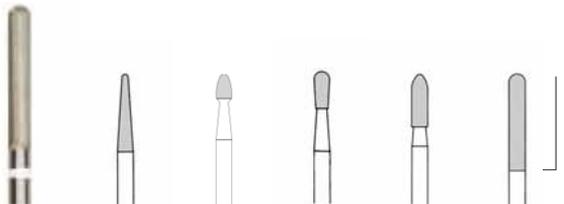
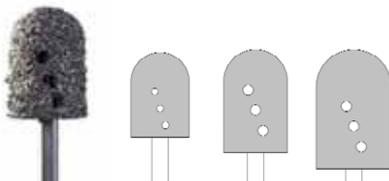
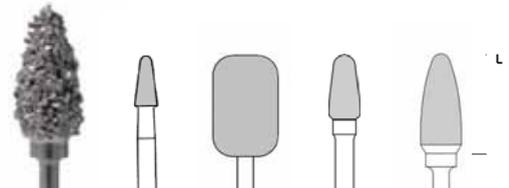
Steel Instruments

MEISINGER steel instruments are manufactured in one piece of vanadium tungsten steel or stainless steel. They demonstrate quality through their functional design, precision toothing and above average concentricity. The instruments offer high elasticity and optimal material hardness. HSS-Steel instruments are produced of high strength steel and therefore offer an additional optimal heat strength, as well as high endurance.

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Advice for sterilization

- Rinse disinfection/cleaning solution (preferably with distilled water)
- Immediately dry instruments thoroughly with absorbent paper or cloth
- Use autoclave pouches according to manufacturers suggestions
- Also note MEISINGER rust preventing spray No. 2820 (page 210)

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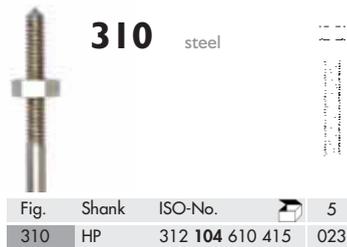
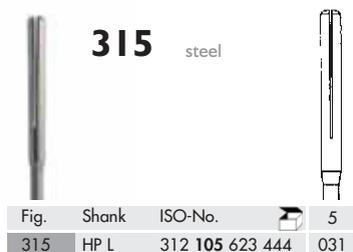
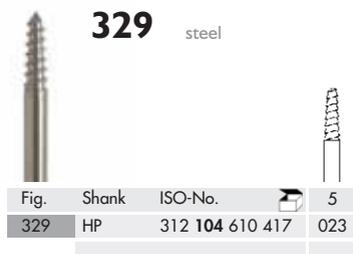
Assortments

MEISINGER sets and assortments were designed and tested by practitioners and leading scientists for a wide range of applications in practice and laboratory. The partially elaborate arrangement of instrument sets is done away with completely. Most of the assortments are delivered in user-friendly and sterilizable bur blocks.

	2413 Page 197	Laminate Veneer Kit
	2600 Page 197	Kinder-Dent-Präpset Children Prep Set Juego odontología infantil
	2666 Page 198	Anterior Veneer and Cementation Kit
	2590 Page 199	Edge Up Präparationsset Edge Up Preparation Kit Kit de preparación Edge Up
	2667 Page 199	Esthetic Inlay/Onlay Kit
	2525 Page 200	Microdia Kit
	2540 Page 200	Plaque Set
Sätze für CAD/CAM • Sets for CAD/CAM		
Freseros para CAD/CAM		
	2606 Page 201	CEREC® Basic Kit
	2605 Page 201	CEREC® Advanced Kit
	2567 Page 202	CEREC® Diamond Kit
	2678 Page 202	Procera® Preparation Kit
	2675 Page 203	Everest® Advanced and Finishing Kit
	2685 Page 203	Cercon® 0°/2°
	2679 Page 204	Procera® Laminate Kit

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CTF Finishing Carbides Set (8 blades), fine

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To trim and finish composite materials. Also for gold and amalgam.

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67 x 67 x 30 mm

2552 CTF-Hartmetallfinierer-Sortiment F (20 Schneiden), extra fein

CTF-Carbide Finishing Set F (20 blades), extra fine

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For finest finishing of filling materials such as composites and cements as well as for optimum smoothing of the cavity margin.

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67 x 67 x 30 mm

2553 CTF-Hartmetallfinierer-Sortiment U (30 Schneiden), ultra fein

CTF-Carbide Finishing Set U (30 blades), ultra fine

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For final finishing of filling materials such as composites and cements as well as for optimum smoothing of the cavity margin.

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67 x 67 x 30 mm

2546 CTF-Diamantfinierer-Sortiment, fein / extra fein

CTF Diamond Finishing Set (Composite Trimming and Finishing), fine / extra fine

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To trim and finish composites, porcelain, and ceramics.

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The HM48L series is the perfect shape for feathering the facial surfaces of anterior teeth. Additionally, the kit includes four varying lengths of finishing burs which allow for greater versatility in finishing and contouring anterior composites. The HM379 egg shaped carbides provide the perfect shape for lingual contouring and finishing. The instruments in this kit are in friction grip (FG) shank.

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IDS 2005 Innovations 2664 Composite Finishing Kit

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The two "football" shaped diamonds 830 and 833 assist in finishing the anterior lingual and posterior occlusal surfaces. The HM246U T&F carbide is perfect for interproximal surfaces while the HM247U is exceptional for posterior occlusion and embrasures. The polishers complete the kit as the 9762 and 9782 are perfect for polishing pits and fissures. The polishers finish the facial surface on anterior teeth.

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This kit contains the essential tools for removing crown and bridge restorations. An extra coarse round and cylinder diamond makes cutting through porcelain fast and efficient. In addition, two sizes of metal cutting carbides (four of each size) are included to effectively cut through metal crowns and substructures. The instruments in this kit are in friction grip (FG) shank.

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IDS 2005 **2635 Comprehensive Diamond Finishing Kit**

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The 820 series is for the removal of excess flash while maintaining contours and embrasures. The diamonds provide varying working lengths which allow for greater flexibility in finishing facial surfaces. The varying lengths are fantastic for easy removal of excess flash, smoothing, and refining of the facial and lingual surfaces. The longer lengths are perfect for line angle to line angle on facial surfaces. The instruments in this kit are in friction grip (FG) shank.

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IDS 2005 **2662 Intra Oral Porcelain Adjustment Kit**

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Like our Composite Finishing Kit (2664) this kit allows for finishing all porcelain restorations. The removal of excess flash and overhang may be accomplished by using the 820 shape diamond. The 890F diamond can be used to refine posterior anatomy or embrasures. The 833F diamond assists with finishing anterior lingual or posterior occlusal surfaces. Additionally, there are two shapes and three grits of diamond impregnated porcelain polishers.

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2595 / 2596 Opti-Shape-Sets – nach Diedrichs

Opti-Shape-Sets – according to Diedrichs

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Assortments for the preparation of crowns, partial crowns, inlays and onlays. Specially developed for galvano-ceramic, metal-ceramic and all ceramic restorations (for example AGC restorations). Edged preparations are avoided by rounded shapes of instruments. Expedient grinding with possibly low roughness by medium diamond instruments and identically shaped finishing carbides.

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An assortment for the preparation of cavities and for finishing composite fillings.

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The new Dia-Set 2531 enables the dentist firstly at the chamfer and shoulder preparation stage for all-ceramics, without the help of special aids, optimal material strength, which achieves a pre-defined ablation. Chamfering with a guide pin: To create a defined chamfer the torpedo type diamond instrument with the axial guide pin is most suitable, because the chamfer is primarily applied circular supragingival, while the polished guide pin, without applying force, is conducted around the tooth. The diamond coated working element leaves an even circular chamfer on the preparation margin. Especially "gutter" type grooves that are raised for the outer preparation margins are in this way avoided.

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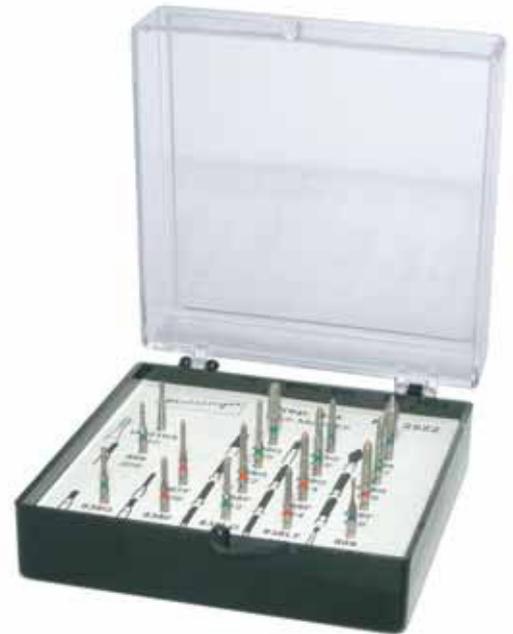
An easy and user friendly preparation set for defined material removal in the area of crowns and inlay preparation.

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Preparation Set – according to Küpper

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Policlinic for dental prosthesis (Director: Univ.-Prof. Dr. Harald Küpper) from University of Jena.

For innovative and conventional crown technology: Glass-ceramic crowns • All-ceramic crowns • Oxide-ceramic crowns • Metal-ceramic crowns

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The 847RH-016 modified shoulder diamond, is used for the axial reduction on pre molars. The 847RH-018 modified shoulder diamond, prepares the axial walls on posterior teeth. For refining the axial walls and margins the HM212LR T&F carbide is the perfect match to the 847RH diamonds. This HM212LR carbide produces smooth axial walls and well defined margins. The 878 shape is excellent for preparing Procera® crowns. The 878-014 size can be used on lateral incisors and pre molars. The instruments in this kit are in friction grip (FG) shank.

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The two depth cutters 834-016 and 834-021 assist with creating pre defined depths on facial surfaces. The smaller creates a 0.3 mm depth on the central incisors and canines, while the larger head size creates a 0.5 mm depth on lateral incisors. The 852 series reduces the facial tooth structure to the pre determined depth. The 852 shaped diamond may also be used on the lingual wrap. The 820 shaped diamond removes interproximal flash and overhang. The 889L and 898 diamonds create an infinite margin and may assist subgingivally. The instruments in this kit are in friction grip (FG) shank.

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2666 Anterior Veneer and Cementation Kit – according to Dr. Stephen Poss D.D.S.

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73 x 25 x 29 mm

The diamond 834-016 is used for minimal facial reduction (0.3 mm) for feldspathic porcelain. The 834-021 produces a 0.5 mm reduction and is used for pressed ceramics like OPC® and Empress I®. The diamond 852H-018 is usually used in conjunction with 834-021 to remove sufficient tooth structure as is shown by the grooves created by the 834 depth cutting diamonds. The 852H-016 is usually used in conjunction with the 834-016. The extra coarse 833H shape is used for the lingual reduction of the central and lateral incisors. It also is used to create a 90 degree margin on the lingual surface. The finishing bur HM375R is used to smooth the facial walls of the preparation. The 839R-012 diamond is used for preparing (smoothing) the external facial margins for pressed ceramics. The HM246-009 (7901) bur is used to separate contacts and to smooth interproximally after cementation. The 833C diamond is used at cementation to blend the ceramic margin to the tooth structure on the lingual margin. The diamond impregnated polishers come in two shapes and three grits, green for adjustment, blue for prepolish and red for producing a high shine lustre.

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Edge Up Preparation Kit – according to Kuntze/Fischer

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90 x 60 x 44 mm

Clinic for Tooth Preservation, Parodontology and Preventive Dentistry, RWTH Aachen, director Prof. Dr. med. dent. F. Lampert.

The Edge Up technique is a new method for restoring defects in the upper and lower anterior regions to produce perfect aesthetics and is as gentle as possible to the tooth structure. This technique is based on a special type of preparation (circumferential extended shoulder preparation in the enamel) and the preparation set is intended specifically for this type of preparation.

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IDS 2667 Esthetic Inlay/Onlay Kit – according to Dr. Stephen Poss D.D.S.

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From the removal of old restorations right up to the final polish this set offers the entire range of instruments for perfect preparation of inlay/onlay construction.

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Diamond instruments as a minimally invasive alternative to oscillating instruments

The microdia kit provides the opportunity to perform minimally invasive techniques in the field of micro-dentistry. Applying this technique provides maximum enamel protection. The rotary diamond instruments distinguish themselves by their small working sections and extremely slim, carved instrument necks and are perfect for minimally invasive preparation techniques. The high definition and the special diamond coating of the instruments permit a preparation with reduced feeding pressure. Suitable especially for microscope-guided work.

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For removing residual plaque from subgingival areas. For treating teeth with two or three roots and severe bone recession which cannot be reached with a curette or scaler. Inaccessible areas should be extended with red ring diamonds. Deposits and calculus should be removed from the root surface with yellow ring diamonds. Best smoothing of root surfaces can be obtained with white ring diamonds. An optimum extra for perfect root surface treatment when treating periodontitis.

Fig.	830LF	830LF	830LC	830LC
Shank	RA L / 205	RA XL / 206	RA L / 205	RA XL / 206
Size	014	014	014	014
L mm	5,0	5,0	5,0	5,0
Pieces	1	1	1	1

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With these especially well-chosen rotating instruments of the CEREC® Kit 2606 a quick and easy dental preparation can be achieved by means of computer-assisted design and milling fabrication. This set contains 6 diamond instruments and 3 carefully selected diamond coated polishers that provide the basis for exceptional crown preparations for CEREC® restorations. Whether performing occlusal reductions, axial wall preparations, interproximal smoothing, or final high shine polishing, all can be achieved with this essential kit.

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In comparison to the Basic-Kit the Advanced-Kit 2605 offers far more application possibilities. For the removal of old crowns, amalgam or other metals, as well as for the initial rough preparation of the tooth, the special crown separator HM31C is most appropriate. The diamond impregnated polishers are great for quickly creating a high shine "wet look." The CEREC® Advanced Preparation and Finish Kit is the one kit that allows you to completely perform all steps necessary to prepare, adjust, and finish your CEREC® restoration quickly, efficiently, and precisely.

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
For contouring the occlusal surfaces of computer cut inlays.

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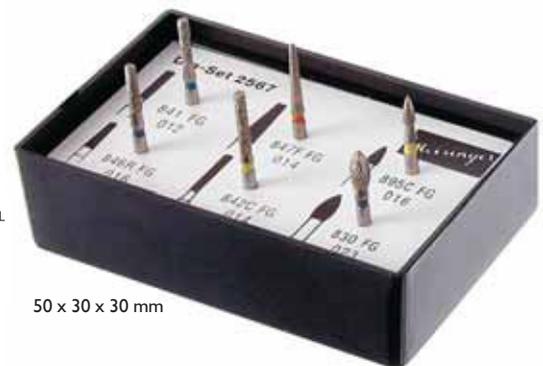
The preparation kit for the Procera® Allceram – Restoration in only a few easy steps to success. The special choice of instruments allows for an optimal anatomic design of the stump. In this case a trimming dimension of 0,8 mm to 1,5 mm should be available.

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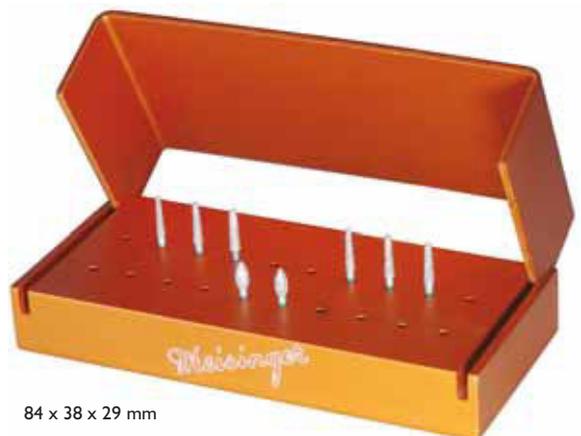84 x 38 x 29 mm

2675 Everest® Advanced and Finishing Kit

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Basis-Preparation Kit for Yttrium-stabilized zirconiumoxide (ZrO₂)-full ceramics and bridges in front and side teeth area. This assortment is ideal and excellent for simple preparation techniques (5-Axial CAD/CAM Systems) in classical stage- or modified chamfer preparation.

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Everest® is a product / trade of Kavo, Germany

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Assortment for Cercon® in CAD/CAM – Technology. Instruments used for the parallel milling technique have an angle of 0° and for the conical milling technique an angle of 2°.

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Cercon® is a product / trade of Degudent, Germany

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Assortment kit for optimal wall thickness structure designs of veneers with Procera® AllCeram. A depth limit of 0,6 mm outlines the ideal material strength.

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2680 Procera® Preparation and Finishing Kit

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
Assortment for designing models and ceramic crowns

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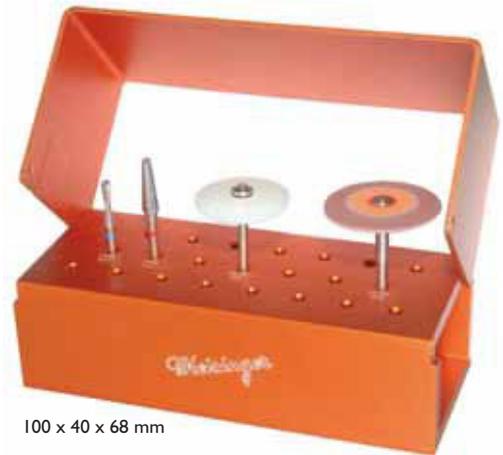Cercon® Crown and Bridge Kit for working on pre-sintered Cercon crowns and bridges

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Zirkon Master Kit for use on zirconiumoxide-restorations. These high-tech materials give pleasure to an ever growing popularity and allow the new zirconium abrasives to be used easily and very precisely. A better removal and with minimal grinding pressure make this an ideal instrument for designing in CAD/CAM-technology.

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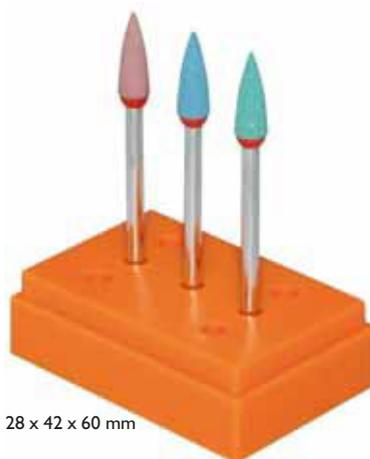
By popular demand we are introducing our diamond impregnated porcelain polishers... in points. The assorted diamond polisher points grant perfect polishing of porcelain crowns and bridges. Greater access for occlusal surfaces to adjust, prepolish and produce a glazed appearance second to none. Another great addition to our ever expanding polishing products. The instruments are HP only.

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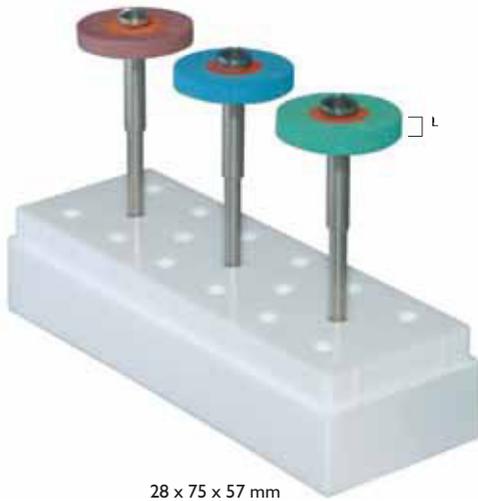
56 x 82 x 71 mm

First there were wheels, then points were requested, and now MEISINGER introduces diamond impregnated porcelain polishers in "knife edge". Color coded like your standard diamonds. Our green/orange polishers are used for adjusting while the blue/orange pre-polish and the pink/orange returns the surface to its desired "wet" glazed appearance without the need of pastes or re-firing. Simply, another great shape in the MEISINGER line of unbeatable porcelain polishers. The instruments are HP only.

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Fig.	9770G	9770M	9770F
Shank	HP / 104	HP / 104	HP / 104
Size	260	260	260
L mm	2,0	2,0	2,0
Pieces	1	1	1

I435 MEIGRIT® Set

HM-Splitter-Fräser-Sortiment

Kit of carbide chip coated cutters - for soft reline removal

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28 x 75 x 57 mm

2515 Prothesen-Korrektur-Set

Carbide Cutter Kit for Denture Adjustments

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The Acrylic Master Kit for use on all types of acrylic, plastic and gypsum. Because of the new facet cutters with cross cut and optimal chip-flute the material can be removed faster and more evenly. A subsequent polishing is herewith made easier.

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A simple and straight forward approach to allow chairside adjustments of acrylic without the problem associated with pumice. Two of our most requested acrylic burs and six acrylic polishers in three grits are included. This entire kit, including bur block, is autoclavable.

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Kit of instruments for shaping and finishing the surfaces of plastic veneer.

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I 320 Titanium Master Kit

developed with University RWTH Aachen

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The tungsten carbide PROFESSIONALS use when working with titanium in dental laboratories and clinics

In cooperation with the Clinic for Prosthodontics of the Technical University Aachen, developed a set of carbide cutters that provide the means of processing different qualities of titanium. The innovative teeth geometries, which range from very fine to coarse, ensure a continually efficient metal reduction in all applications and a high quality surface. The parallel grinding instruments are suitable under dry conditions as well as with the use of cutting oil. The design of the carbide cutters enables a smooth running and guarantees a high service-life even under high strain conditions. There is a set of polishers integrated in the tool-kit which care for a high-end surface finishing.

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The UNTOUCHABLES for working with titanium

The new carbide cutters ST are distinguished by their recently developed super-coarse cutting geometry and their extremely quick trimming.

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Accessory

Accesorios

MDC 100 Disinfection and Cleaning Solution

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The convenient and economical solution for cleaning and disinfecting the complete range of medical and dental instruments.

The application of the MDC 100 is extremely user-friendly because of the packaging of pouches that list an advised mixing ratio of the new concentrate. The convenient pouch packaging helps to eliminate the need for costly storage space that large volume containers require and also eliminates the unnecessary remaining quantities often seen with these containers.

For the application of the MDC 100, just pour and dissolve the contents of a single pouch into the provided matching container, filled with 900 ml of water. The validated mixing proportion is achieved fast and simply which guarantees an ideal cleaning and disinfecting of the instruments. The shape of the container offers ample room for cleaning individual instruments as well as complete bur stands.

MDC 100 distinguishes itself by its high cleaning strength and its outstanding corrosion prevention. It is especially fast and effective and at the same time offers a high material compatibility. It enables the cleaning of the whole dental instrument range without unpleasant odors and, in addition, when dissolved in water, is entirely biodegradable.

Conclusion: Open pouch. Pour on water. Ready!

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Dosage: (in accordance with the new*DGHM-Guidelines):

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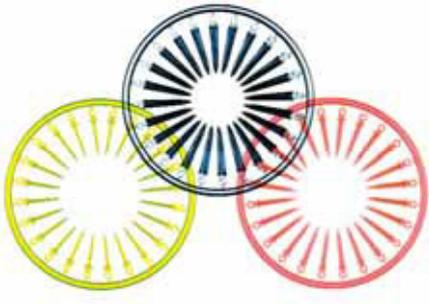
The MEISINGER rust preventing spray prevents rusting during and after sterilization. Applicable for all methods of sterilization (autoclave: max. 416 K (143 °C), hot air sterilization: max. 453 K (180 °C)). Easy to use, excellent results.

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Interdental wedges – according to Dr. Grabosch

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34401:

5 Ringe à 24 Keile Gr. 1, gelb (klein)

5 circles of each 24 wedges of size 1, yellow (small)

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34402:

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5 circles of each 24 wedges of size 2, red (medium)

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34403:

5 Ringe à 24 Keile Gr. 3, blau (gross)

5 circles of each 24 wedges of size 3, blue (big)

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Bur cleaning brush, scrubbing brush, brass

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Assorted box of 2 x size 1 (yellow), 2 x size 2 (red), 1 x size 3 (blue):

- functional
- optimum handling
- colour codification
- fixed in a circular arrangement made of fluorescent POLYCARBONATE
- free of cadmium elasticity moulds for optimum adaptability
- security by preventive thread and loop
- autoclavable

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Bur cleaning brush, scrubbing brush, nylon

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These products are manufactured from coloured anodised aluminium. When using chemical cleaning solutions a slight discolouring may occur. Therefore when using cleaning solution (recommended pH-value 4-8) it is advisable not to use those which have high contents of alkaline or acid.

The following sterilisation method can be used without hesitation: Sterilising with humid heat in the autoclave (e.g. at 134°C). Please follow the operating instructions of the appliance according to the manufacturer and observe the recommendations regarding the duration of sterilisation.

The following application tips apply for aluminium products:

- Do not use thermo disinfectant
- Do not use an ultrasonic bath
- Observe the regulations precisely of the disinfectant manufacturer: some are not suitable for aluminium!

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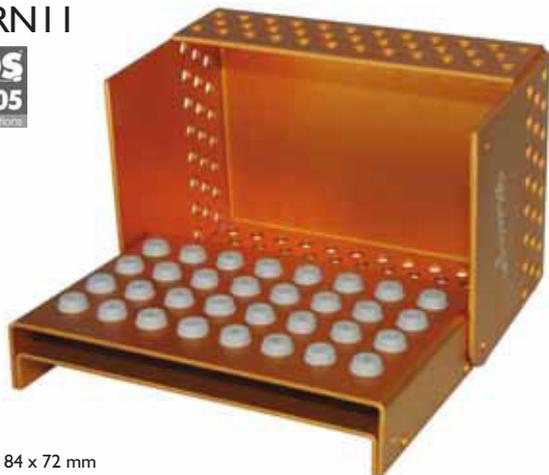
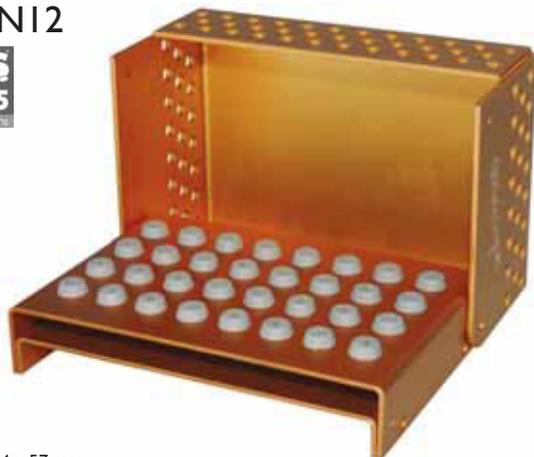
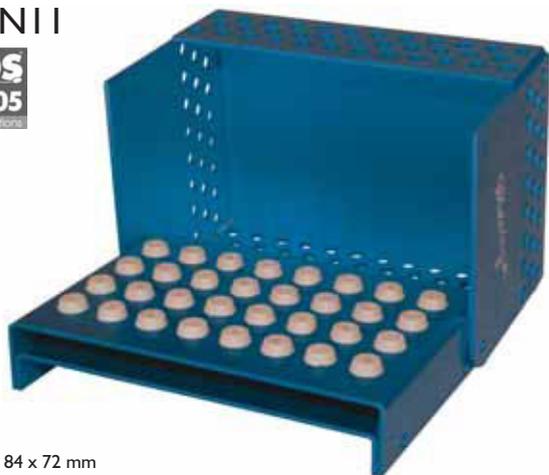
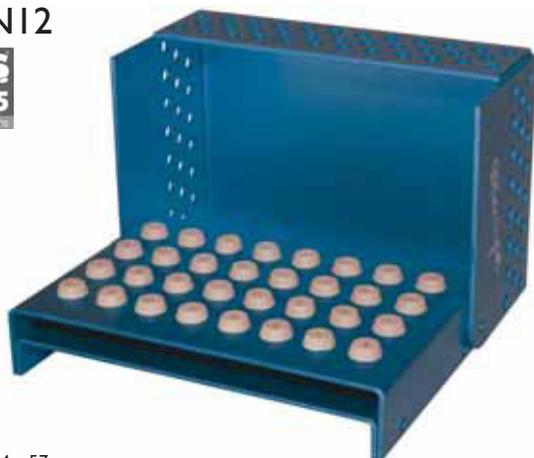
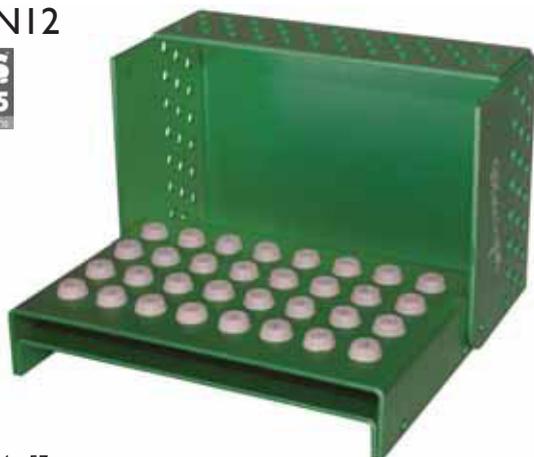
Base - For 50 handpiece or RA instruments

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Base - For 25 handpiece or RA instruments and 25 FG-instruments

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General application and safety instructions for the medical device produced by MEISINGER

Issued: January 2008

- MEISINGER medical products (dental, implantology, Bone Management®, maxillary surgery, general surgery, chiropody devices) must only be used by dentists, doctors and/or the respective experts who, due to their training and experience, are intensely familiar with the use of these products and who have the corresponding expertise in the respective specialist fields. The use of surgical products requires relevant expertise and experience in dental implantology, maxillary surgery and/or other surgical fields including diagnosis, preoperative planning and surgical techniques.
- It is the sole responsibility of the doctor in charge who, depending on the respective situation (indication), decides on the actual use of the MEISINGER products for each individual case.
- Courses providing further training on the possible uses of MEISINGER products are organised on a regular basis. You will find relevant information about these events as well as further information on MEISINGER products on the internet at www.meisinger.de.
- All Meisinger products have been developed for specific applications. Therefore, inappropriate use can result in the premature wear and tear of the products and put patients and users at risk.

Application

- It is essential to only use turbines as well as hand and angle pieces that are technically and hygienically faultless, maintained and cleaned.
- Rotating instruments need to be clamped as far down as possible with their speed set before applying them on the object. are used with the rotary instruments.
- Using the instruments for canting or leveraging should be avoided as it increases the risk of breakage.
- Depending on the application, it is recommended to use protective goggles while using the instruments. Users of diamond disks should use a disk protection device.
- Inappropriate use of the products leads to badly executed work and increased risk.
- When working with dry materials, it is recommended to use a suction cleaning device.
- In particular, users of hand tools should take care to use them gently and with consideration.
- The user must at all times avoid touching the instruments and parts without protection (protective gloves should be worn).
- Thermal bone damage caused by rotating tools (e.g. pilot burr, conical burr, expansion burr) must at all times be avoided (user training, working at low speed and with sufficient cooling).

Use of pressure

- Users of the instruments should at all times avoid applying excessive pressure. This can damage the working part of the instruments and cause the cutting edges to break off. At the same time, it generates excessive heat.
- The use of excessive pressure when using grinding tools can cause the abrasive particles to break off or the instrument to become clogged and lead to heat generation.
- During polishing, excess pressure can lead to heat generation.
- Due to overheating, excess pressure can damage the dental pulp or, due to broken off cutting edges, it can result in undesired rough surfaces. In such cases, even instrument breakage cannot be excluded.

Cooling

- In order to avoid excessive heat generation during preparation, a sterile water/sodium chloride solution supplied via a permanent external feeding device should be used to ensure sufficient cooling during use of the instruments.
- When using FG instruments that are more than 22 mm long or whose head diameter exceeds 2 mm, additional external cooling is required.
- Insufficient cooling will lead to irreversible damage to the bone and/or the adjacent tissue.

Storage, disinfection, cleaning and sterilization

- Unless there is explicit information to the contrary, all MEISINGER products are supplied in non-sterile packaging and, depending on the application, they need to be sterilised prior to use. Prior to their first use on the patient and immediately after each use, all products need to be disinfected and sterilised. Inappropriate cleaning and sterilising of the instruments can result in the patient being infected with harmful bacteria.
- You will find detailed instructions for the disinfecting, cleaning and sterilising of products in the Instructions for the reprocessing of medical devices produced by Hager & Meisinger GmbH. We would be happy to provide you with these instructions at your request. They are also available on the internet at www.meisinger.de.
- The products should be stored in appropriate, hygienically maintained containers. The same applies to sterilised instruments. The stored products must be protected from dust, humidity and recontamination. Instructions as to maximum storage duration must be adhered to.

Speed recommendations

- Following the instrument-specific speed recommendations produces the best results.
- Exceeding the maximum admissible speed (rpm) when using long and pointed instruments tends to produce vibrations that can lead to the destruction of the instrument.
- When using working parts with diameters exceeding the thickness of the shaft, excessive speed can release great centrifugal forces that may cause the shaft to bend and/or the instrument to break. Therefore, the maximum admissible rpm must never be exceeded.
- Please consult the manufacturer's information (see catalogue or www.meisinger.de) for the recommended and the maximum admissible speed ranges. Non-compliance with the maximum admissible speed puts safety at risk.
- Generally, the following rules apply:

– The larger the working part of an instrument the lower the speed

- The larger the working part of an instrument the greater the force of pressure
- Instruments with a maximum speed of 300,000 rpm are suitable for micro-motor hand pieces and turbines with stable ball bearings. Not to be recommended for turbines with air bearings.
- Instruments with a maximum speed of 30,000 to 160,000 rpm are suitable for micro-motor hand pieces or technical hand pieces up to the indicated rpm. Not to be recommended for turbines.
- Surgical instruments: suitable for geared down micro-motor hand and angle pieces 1:10 with stable ball bearings. Speed 600 to 800 rpm with physical and, possibly, sterile external cooling or internal cooling when using the respective hand piece.

Discarding worn instruments and parts

- As a rule, MEISINGER medical products are re-usable unless there is explicit proof and information to the contrary. It is the sole responsibility of the doctor using the devices to decide on the possibility of and the justification for multiple use as well as on the frequency of use, based on the respective case and the potential wear and tear of the products. If in doubt, it is always best to discard the products early and to replace them.
- Instruments are subject to wear and tear. This is particularly true for rotating tools. Therefore, our warranty for all rotating tools expires after their fifth use. If the tools are used more frequently, the manufacturer, Hager & Meisinger GmbH, can no longer guarantee a faultless cutting performance combined with minimum temperature change.
- Broken off cutting edges of instruments cause vibrations and great forces of pressure, which, in turn, leads to broken preparation corners and rough surfaces.
- Bare patches on diamond instruments indicate a lack of abrasive particles and can be a sign of blunt instruments. This leads to excessive temperatures during instrument use.
- Instruments that are bent and/or do not run true should be discarded forthwith.

Additional instructions for the use of implants

- As a rule, implants are meant to be used only once.
- Implants must only be inserted by dentists and doctors who are intensely familiar with dental implantology including diagnosis, pre-operative planning, surgical techniques and prosthetic care.
- The unprotected touching of the implant by the user must be avoided (protective gloves must be worn).
- Systemic errors that are due to a mix-up of tools and implants are to be avoided. Therefore, paying attention to colour coding and labelling is essential.
- A lack of primary stability can be due to the faulty preparation of the implant bed as well as to deficient bone quality. In such cases, the mobile implant should be removed and replaced with a larger one.
- The best way to avoid complications is to only use implants when all the conditions for a complication-free treatment guaranteeing long-term success have been met. If, from the start, there are reservations regarding the chances of success of a desired implant-based dental prosthesis, it may be best to choose another prosthetic solution that does not require an implant.

Additional instructions regarding the use of trepanen

- When using trepanen, you have to proceed with particular care. For example, it is advisable not to exceed the recommended rpm speed ranges.
- In order to prepare for the actual use of a trepan, it should be set to produce counter-clockwise rotations creating a groove in the bone. Afterwards the trepan can be inserted into this groove and, using clockwise rotations, it can be moved further down.
- Carrying out a prior X-Ray is essential to establish the maximum possible drilling depth and to maintain the necessary distance, for example, to the mandibular nerve. As an additional safety measure to spare the nerve, the axial direction of the trepan countersink attachment, based on the sagittal level of the ascending branch, must be milled laterally at an angle of approx. 15-20°.

Further comments:

- Due to statutory regulations, returned goods can, on principle, only be accepted if the complete batch number is provided. This number can be found on the product packaging.

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Instructions for the reprocessing (cleaning, disinfecting and sterilising) of medical devices produced by Hager & Meisinger GmbH

Issued: January 2008

The medical devices produced and sold by Hager & Meisinger GmbH are re-useable unless their label contains explicit information to the contrary. However, as a rule, it is the sole responsibility of the doctor/expert using the devices to decide whether, depending on the respective case and the potential wear and tear of the products, he can re-use the products and how frequently he uses them. In case of doubt, it is always advisable to discard the products early and to replace them. The manufacturer Hager & Meisinger GmbH cannot guarantee the faultless function and performance of the products combined with a maximum degree of safety if the products are overused. These reprocessing instructions apply in principle to all medical devices making up the product range supplied by Hager & Meisinger GmbH. Any particular features and/or exclusions that only concern individual items or groups of items are referred to separately. As to the general application and safety instructions concerning the use of the products, we would advise you to consult the application and safety instructions for the medical products of Meisinger, which are available separately (see also at www.meisinger.de).

General rules

All products need to be cleaned, disinfected and sterilised prior to use; this applies in particular to the first-time use of products i.e. after their delivery because all products are supplied non-sterile (cleaning and disinfecting after the removal of transport packaging; sterilisation after removing wrapping). The thorough cleaning and disinfecting is indispensable in order to achieve effective sterilisation.

As you are responsible for the sterility of the products during use, please ensure

- that, as a rule, only sufficiently validated device and product-specific procedures are used for the cleaning/disinfecting and sterilisation process,
- that the used devices (disinfectant, sterilizer) are maintained and checked at regular intervals and
- that the validated parameters are adhered to during each cycle.

Please ensure that, during use, you separate the soiled products and do not return them to the bur block/instrument tray to avoid further contamination of the loaded bur block/instrument tray. After cleaning/disinfecting the soiled products, arrange them in the bur block/instrument tray and sterilise the fully loaded bur block/instrument tray. Please also take note of the statutory regulations valid in your country as well as the hygiene rules followed by the doctor's practice or the hospital. This applies in particular to the different rules regarding effective prion deactivation. You will find further instructions for the reprocessing of medical products and information on suitable products to carry out the reprocessing on the internet at www.dghm.de or at www.rki.de. As the products are destined to be used for surgical, parodontological or endodontic procedures such as root canal debridement, they may penetrate the skin or the mucosa and come into contact with blood, internal tissues or organs (including wounds). Therefore, we recommend that they be assigned to risk group Critical B if used for their intended purpose. Attention: For some products, additional aspects need to be considered (see chapter on 'Special instructions')!

Cleaning and disinfecting

Basic rules:

Ideally, a mechanical procedure (disinfectant) should be used for the cleaning and disinfecting process. Due to their clearly reduced effectiveness and reproducibility, manual procedures – even if they include the use of an ultrasound bath – should only be used if an automated procedure is unavailable. Pre-treatment needs to be carried out in both cases.

Apply hygiene-effective measures in line with the RKI guidelines at your workplace and carry out all work using powder and latex-free gloves.

Pre-treatment:

Abrasive impurities need to be removed from the products directly after use (within two hours maximum). To do so, use running water or a disinfectant solution; the disinfectant must not contain aldehydes (which could fix blood residues to the instrument surface), its effectiveness should be established (e.g. it should be certified by the VAH (German Association for Applied Hygiene), the DGHM (German Society for Hygiene and Microbiology) or the FDA or have CE marking), it should be suitable for the disinfection of the products and compatible with the products (see chapter on 'Material resistance'). For the manual removal of impurities, only use nylon brushes intended for the purpose (also see the chapter on 'Special Instructions') or clean soft and lint-free cloths that you only use for this purpose. Do not use metal brushes or steel wool.

As to products with lumen (cavities): rinse all cavities three times by using a disposable syringe (minimum volume 5-10 ml) and a cannula.

Please note that the disinfectants used during pre-treatment only ensure personal protection and can be no substitute for the disinfection procedure to be used later - after completion of the cleaning process.

Mechanical cleaning/disinfection (disinfectant/RDG (cleaning and disinfection device):

- When choosing a disinfectant you will have to ensure
- that the effectiveness of the disinfectant has been certified (e.g. it has been licensed by the DGHM or the FDA or has CE marking according to DIN EN ISO 15883),
- that, if at all possible, a programme is used that has been certified for thermic disinfection (A0-value > 3000 or – with regard to older devices – at least 5 minutes at 90 °C (194 °F)) (chemical disinfection runs the risk of disinfectant residues remaining on the instrument),
- that the used programme is suitable for the products and has a sufficient number of rinsing cycles,
- that only sterile water or water with low levels of germs (max. 10 germs/ml) and endotoxins (max. 0.25 endotoxin units/ml) is used for the post-purge cycle (e.g. purified water/highly purified water),
- that the air used for drying is filtered and that the disinfectant is regularly maintained and checked.

When choosing an appropriate cleaning system you need to ensure:

- that it is generally suitable for the cleaning of products made of metal and plastic,
- that, in addition, – if no thermic disinfection is used - a disinfectant is used whose efficacy has been certified (e.g. VAH/DGHM FDA licence or CE marking) and
- that it is compatible with the used cleaning agents and that the used chemicals are compatible with the products (see chapter on 'material resistance').

It is essential that the concentrations recommended by the manufacturer of the cleaning and the disinfectant agent (if required) are adhered to at all times.

Procedure:

1. Disassemble the products as far as possible.
2. When using products with lumen: ensure that all lumens are rinsed effectively as part of the pre-cleaning process.
3. Place the disassembled products in the disinfectant. Take care that the products do not touch. If possible: connect all
4. Start the programme.
5. Remove the products from the disinfectant after the programme has finished.
6. Check and wrap the products straight after removal if possible (see chapters on 'Checking', 'Maintenance' and 'Packaging') if necessary, after they have been dried off completely in a clean place.

Proof of the general suitability for effective mechanical cleaning and disinfecting has been provided by an independent certified test laboratory using the disinfectant G 7836 GD (thermic disinfection, Miele & Cie. GmbH & Co., Gütersloh) and the cleaning agent Neodisher medizym (Dr. Weigert GmbH & Co. KG, Hamburg). As part of the testing the laboratory used the above described procedure.

Manual cleaning and disinfection:

When choosing an appropriate cleaning and disinfecting agent you need to ensure

- that they are generally suitable for the cleaning and/or disinfection of products made of metal and plastic,
- that the cleaning agent, where used, is suitable for ultrasound cleaning (no production of foam)
- that an appropriate disinfectant is used whose efficacy has been certified (e.g. VAH/DGHM FDA licence or CE marking) and that it is compatible with the used cleaning agent and
- that the used chemicals are compatible with the products (see chapter on 'Material resistance').

Ideally, combined cleaning/disinfecting agents should not be used. Combined cleaning/disinfecting agents can only be used in cases where there is a very low degree of contamination (no visible soiling).

It is essential that the concentrations and contact times recommended by the manufacturer of the cleaning and the disinfectant agents are adhered to at all times.

Only use freshly made solutions and water that is either sterile or low in germs (max. 10 germs/ml) and endotoxins (max. 0.25 endotoxin units/ml) (e.g. purified water/highly purified water) and only use filtered air for drying.

Procedure: Cleaning

1. Disassemble the products as far as possible.
2. Place the disassembled products in the cleaning bath for the prescribed contact time and ensure that products are sufficiently submerged; where required, use ultrasound support but at least careful brushing using the nylon brushes intended for this purpose (see also chapter on 'Special Instructions'). Take care that the products do not touch. With regard to products with lumen: rinse all instrument lumens five times at the beginning and or at the end of the contact time using a disposal syringe (minimum volume 5-10 ml) and a cannula.
3. Afterwards, take the products out of the cleaning bath and rinse at least three times thoroughly with water. With regard to products with lumen: rinse all instrument lumens five times at the beginning and or at the end of the contact time using a disposal syringe (minimum volume 5-10ml) and a cannula.
4. Check the products (see chapters on 'Checking' and 'Maintenance').

Disinfection:

5. Place the disassembled, cleaned and checked products in the disinfecting bath for the prescribed contact time and ensure that the products are sufficiently submerged. Take care that the products do not touch. With regard to products with lumen: rinse all instrument lumens five times at the beginning and or at the end of the contact time using a disposal syringe (minimum volume 5-10 ml) and a cannula.
6. Afterwards, take the products out of the disinfecting bath and rinse them thoroughly with water at least three times. With regard to products with lumen: rinse all instrument lumens five times at the beginning

and or at the end of the contact time using a disposal syringe (minimum volume 5-10 ml) and a cannula.

7. Dry the products by blowing them dry using filtered compressed air.

8. Wrap the products, if possible, straight after removal (see chapter on 'Packaging', if necessary, after they have been dried off completely in a clean place).

Proof of the general suitability for effective manual cleaning and disinfecting has been provided by an independent certified test laboratory using the cleaning agent Gigazyme and the disinfectant Gigasept Instru AF (Schülke & Mayr GmbH, Norderstedt). As part of the testing, the laboratory used the above described procedure.

Checking

After all products have been cleaned and/or cleaned/disinfected, check them for corrosion, damaged surfaces/bare patches, broken/chipped-off edges, deformations (e.g. bent rather than round) and impurities and eliminate damaged products (limited numbers for re-use see chapter on 'Re-use'). Products that are still contaminated need to be cleaned and disinfected once more.

Maintenance

Re-assemble disassembled products (see specific instructions). Instrument oils must not be used.

Packaging

Arrange the cleaned and disinfected products in the dedicated bur block/sterilisation tray. Wrap the products and/or the bur blocks/sterilisation trays using disposable sterilisation packaging (disposable or double packaging) and/or sterilisation containers that meet the following requirements:

- DIN EN ISO/ANSI AAMI ISO 11607
- Suitable for steam sterilisation (thermal stability up to 137 °C (279 °F) sufficient steam permeability)
- Sufficient protection of the products and/or sterilisation packaging against mechanical damage
- Regular maintenance according to the manufacturer's specifications (sterilisation container)
- Individual packaging: the packaging must be sufficiently large to ensure that the sealing is tension-free.

Sterilisation

We only recommend the use of the sterilisation procedures listed below. Steam sterilisation

- Use of a fractional vacuum process or a gravitation process! (with sufficient product drying)
- Steam sterilizer according to DIN EN 13060 and/or DIN EN 285
- Validated according to DIN EN ISO 17665 (up to now: DIN EN 554/ANSI AAMI ISO 11134) (valid IQ/OQ (commissioning) and product-specific performance assessment (PQ))
- Maximum sterilisation temperature 134 °C (273 °F; plus tolerance according to DIN EN ISO 17665 (up to now: DIN EN 554/ANSI AAMI ISO 11134))
- Sterilisation time (exposure time at the sterilisation temperature)
 - at least 20 min at 121 °C (250 °F) and/or at least 5' min at 132 °C (270 °F)/134 °C (273 °F)
 - 1 Use of the less effective gravitation procedure is only admissible if the fractional vacuum process is unavailable.
 - 2 or 18 min (prion deactivation)

Proof of the general suitability of the products for effective steam sterilisation has been provided by an independent certified test laboratory using the steam sterilizer Systec V-150 (Systec GmbH Labor-Systemtechnik, Wetztenberg) as well as the fractional process and the gravitation process. During the testing, the laboratory took into account the typical hospital and practice conditions as well as the procedure described above. As a rule, the flash sterilisation procedure is not admissible. Furthermore, do not use hot air sterilisation, radiation sterilisation, formaldehyde or ethylene oxide sterilisation or plasma sterilisation. In order to avoid stains and corrosion, the steam must be substance-free (see limit values included in DIN EN 13060). When sterilising several devices, the maximum load of the sterilising apparatus must not be exceeded (observe the manufacturer's instructions).

Storage

Prior to the first use of the device, the product should be stored in its original packaging at room temperature in dust- and humidity-free conditions. Subsequently, the products should be stored in appropriate hygienically maintained containers (protected from dust, humidity and recontamination). After sterilisation, the products need to be stored in sterilisation wrapping in a dry and dust-free place. Please note the shelf-life resulting from the validation of the sterilisation wrapping.

Material resistance

When choosing the cleaning and disinfecting agents ensure that they do not contain the following components:

- organic, mineral and oxidising acids (minimum admissible pH value 5.5)
- strong alkaline solutions (maximum admissible pH value 8.5, neutral/enzymatic cleaning agent recommended)
- organic solutions (e.g. alcohols, ether, ketones, benzines)
- oxidants (e.g. hydrogen peroxides) - halogens (chlorine, iodine, bromine)
- aromatic/halogenated hydrocarbons

Never clean products, bur blocks and sterilisation trays using metal brushes or steel wool. Products, bur blocks and sterilisation trays should never be exposed to temperatures exceeding 137 °C (279 °F)!

Re-use

When it comes to products that are re-used, the end of their useful life tends to depend on their wear and tear as well as the use-related damage. Therefore, it is necessary to check the instrument after or prior to each re-use (see section on 'Checking'). Independently of this, it is the sole responsibility of the physician using the instrument to decide upon re-use based on the respective case as well as the potential wear and tear of the instrument. In this context, known restrictions as to the frequency of instrument use need to be observed. If you fail to observe them, we do not accept any liability.

Special instructions

Benex-Control (Extractor):

- Disassemble instrument completely (including the Teflon plate) prior to cleaning and disinfecting
- Reassemble instrument prior to wrapping and sterilisation; ensure that the ball does not catch in a cavity of the biteplate (semi-tilted position)

Benex-Control (Traction cable):

- Bend cable during pre-cleaning and manual cleaning and disinfection several times in all directions

Drill extension:

- Disassemble instrument completely prior to cleaning and disinfecting
- Reassemble instrument prior to wrapping and sterilisation

Diamond products and ceramic grinding tools:

- Use particular care during the cleaning of the grinding surfaces and ensure that all residues are removed

Distractors:

- Move instrument up and down several times during pre-cleaning and manual cleaning and disinfecting
- Mechanical cleaning/disinfection as well as sterilisation only in specific positions (VCD05: half-open state, VCD15: nearly closed state)

Products equipped with a conduit to supply a cooling medium and other products with lumens (canals, drillings etc.):

- Actively rinse the lumen during pre-cleaning and manual cleaning and disinfection process
- Products containing lumens without feed-through channels must not be re-used

Supports:

- Disassemble instrument completely prior to cleaning and disinfecting
- Wrapping and sterilisation only in disassembled state.

Bur blocks/instrument trays:

- Cleaning and disinfecting only without products being loaded (products must not be cleaned and disinfected whilst they are in the bur block/ instrument tray)
- Remove attachments prior to cleaning and disinfecting and clean and disinfect in disassembled state

Universal Torque Ratchet:

- Disassemble instrument completely prior to cleaning and disinfecting
- Move joint several times to and fro during pre-cleaning and manual cleaning and disinfecting

Reassemble instrument prior to wrapping and sterilisation.

The above instructions have been validated by the manufacturer of the medical devices who found them to be SUITABLE for preparing a medical device for re-use. It is up to the person in charge of the reprocessing to ensure that, based on the use of the correct equipment, material and personnel in the reprocessing facility, the actual reconditioning process produces the desired results. Normally, this requires the validation and routine monitoring of the procedure. Equally, each deviation from the instructions provided should be carefully checked for effectiveness and potential adverse consequences by the person in charge of reprocessing.

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